

Application

Step 1



Peel away the white/orange backing papers marked 1

Step 2



Holding the non-adhesive tabs at the end of the dressing, carefully position the dressing over the designated area.

Step 3



Smooth the dressing into place, applying light pressure (especially around edges).

Step 4



Using the small tab, begin to gently peel off the top carrier marked 2. Smooth the dressing down. Ensure your patient is comfortable with the dressing.

Removal



- 1 Lift one corner.
- 2 Pull towards you - away from the pad.
- 3 Release.
- 4 Repeat along the edge of the dressing.

When to change



No change necessary



No change necessary



Change dressing

Support pack available including:

- Application posters
- 'When to change' posters
- Ward training sessions

Wound Management
Smith & Nephew
Healthcare Limited
Healthcare House,
Goulton Street,
Hull HU3 4DJ.

T 01482 222200
F 01482 222211

advice@smith-nephew.com
www.smith-nephew.com


Customers in Ireland please
call T 1890 224 226
for product enquiries and
wound care advice

*Trademark of Smith & Nephew
© Smith & Nephew Jan 2008
12087

References

- 1 Hammond, V; DS/07/209
- 2 Tompkins, L; DS/07/115
- 3 Report reference WRP-TW042-281 Bacterial barrier testing of IV3000

* When compared to OPSITE Post-Op

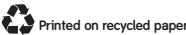

OPSITE Post-Op Visible Ordering Codes

Code	Size	Carton	NHS Codes
66800137	15cm x 10cm	20	ELW227
66800138	20cm x 10cm	20	ELW228
66800139	25cm x 10cm	20	ELW229

 **smith&nephew**
OPSITE[®]
POST-OP VISIBLE
Waterproof, Bacteria-proof
Dressing with See-through
Absorbent Pad

The future of post-operative care is clear to see

Open your eyes to a more absorbent^{*1}, bacteria-proof¹ dressing that allows clear visibility of your surgical wounds.




Clearly better protection

After all the hard work in the operating theatre to prevent infection, keeping the site free from infection must be a high priority. One solution is to use a post-operative dressing that will keep infection out.

OPSITE[®] Post-Op Visible allows you to do this, as well as offering superior absorbency^{*1,2} and the visibility of the wound.


Infection protection

- The unique film of OPSITE Post-Op Visible is bacteria-proof³. The moisture responsive film helps keep the site dry¹. This helps keep the dressing securely in place to seal the site.
- The hydrocellular lattice foam pad allows you to see the wound at all times to identify any early signs of infection and avoiding the need to lift the dressing to inspect the wound

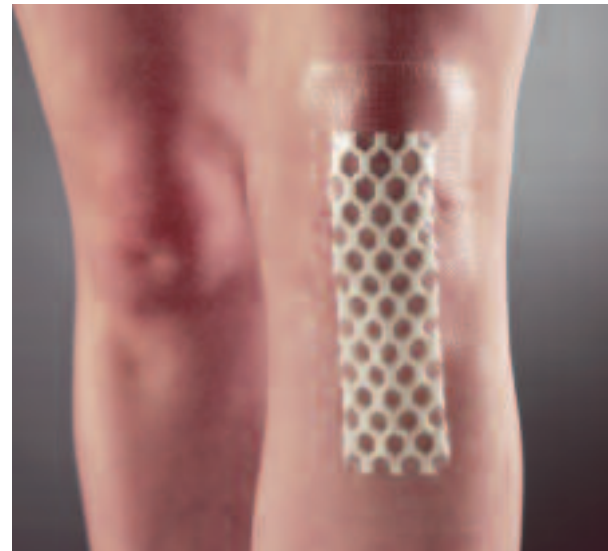

Optimum healing environment

- OPSITE Post-Op Visible may create an optimum healing environment as the pad absorbs excess wound discharge and the high MVTR film allows excess moisture to transpire away
- OPSITE Post-Op Visible will handle more exudate than leading post-operative dressings^{1,2}.

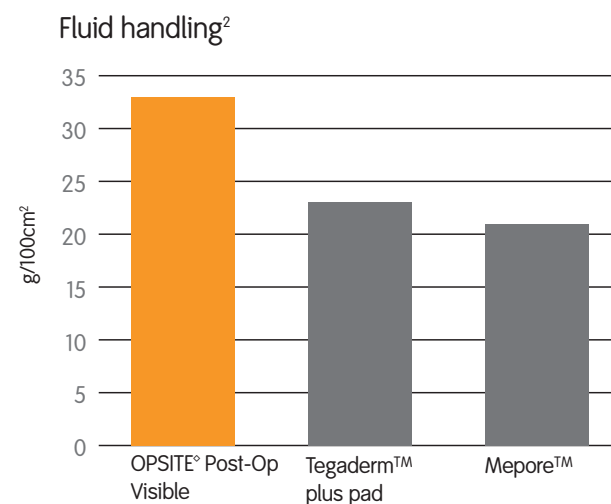

Visibility

- The innovative design of the hydrocellular foam pad allows you to monitor the surgical site without the need to change the dressing. It gives you the opportunity to identify early signs of infection and take prompt action.

OPSITE Post-Op Visible is an innovative post-operative dressing, which combines superior absorbency^{1,2} with the ability to monitor the wounds progress, without lifting the dressing or disturbing the wound

Key Characteristics

OPSITE Post-Op Visible has been designed especially for low to moderate exuding post-operative wounds, lacerations, cuts, abrasions and minor burns.

